



WEB

www.ossur.co.uk

E-MAIL

ossuruk@ossur.com

TEL

08450 065 065

FAX

0161 490 8501

Knee

Introduction

Össur's comprehensive ligament knee braces have been developed over more than 25 years. They provide high levels of control for a wide variety of applications from prophylaxis to recovery post injury.

Whether you need a post operative brace, a semi-rigid brace for lower impact activities, or a more rigid brace for high contact sports, Össur has a premium quality solution for you.

Many people suffer unnecessarily from knee pain due to osteoarthritis (OA), a degenerative joint disease characterised by breakdown of the joint's cartilage that cushions the bones of the knee. Össur's OA knee braces offer a non-invasive, economical treatment option that is scientifically proven to relieve pain and improve mobility.

We offer a wide selection of products including the revolutionary Unloader One®, the CTi® brace and Bio Skin® supports.

Our Selection

Knee Ligament

CTi® Custom.....	92
CTi® OTS	95
Flex™	98
Paradigm® OTS	99
Trainer™	100

Knee OA

Unloader One® Custom	102
Unloader One® OTS.....	104
CTi® OA	107
OAsys® Carticare.....	109

Post-Op Knees

Innovator DLX®	111
Innovator®	112
Advance® Knee	113

Knee Immobiliser

Exoform® Knee Immobiliser	115
Universal 3-Panel Knee Immobiliser.....	116
DeLuxe Knee Immobiliser.....	117

Knee Supports

Bio Skin® Gladiator	119
Bio Skin® Hinged Knee Skin™.....	120
Bio Skin® Standard Knee Skin™	121
Form Fit® Knee Brace	122
ThermoWrap™	123

Knee Patella

Bio Skin® Q Lok Brace.....	125
Bio Skin® Q Brace™	126
Bio Skin® Patella Stabiliser.....	127
Bio Skin® Visco Knee Skin™	128
Bio Skin® Q Baby™ Patella Tendon Strap	129
Neoprene Knee Support with Stabilised Patella...	130

Sleeves

Bio Skin® Under Skin™	132
Bio Skin® Thigh Skin™	133
Bio Skin® Calf Skin™	134
Bio Skin® Shinsplints Wrap/Band	135

Made-to-measure products

Specials - Custom Thermowrap.....	136
CTi Custom/Unloader One Custom Order Form.	137

Knee Ligament

CTi® Custom.....	92
CTi® OTS	95
Flex™	98
Paradigm® OTS	99
Trainer™	100

CTi® Custom

Introduced in 1983, CTi® remains the cornerstone of our ligament bracing product line. While the CTi® concept has evolved over time, our core philosophy remains unchanged, a static support system consisting of a rigid frame that will hold the bones in correct alignment and provide high levels of control to prevent displacement. The constant development and improvement of this classic design is why CTi® remains the top choice of athletes and leading clinical specialists around the world.

The hand laminated, rigid, carbon composite CTi® frame captures the tibia, providing the brace-to-bone contact needed for maximum stabilisation of the knee joint.

The Accutrac™ hinge was developed by analysing natural leg motion and resulted in the invention of a patented 4-bar linkage system, that more accurately mimics the natural movement of an unbraced knee.

Össur Sensil® Silicone Technology was initially developed for the prosthetics industry and has now been integrated into the CTi®. In combination with an Aerospacer base mesh fabric, this innovative and patent pending breathable lining material virtually eliminates brace migration.

Skin-friendly SensEdge overmoulded flexible D-rings provide comfort and elimination of any potential pressure points. They are attached internally for a more streamlined low profile fit as well as providing improved anchorage to minimise any movement.

The CTi® Custom is generally used for high levels of support where comfort and strength are not compromised by weight and bulkiness. It can also offer greater advantages for accurate fitting where leg shape abnormalities present a problem.



CTi® Custom

INDICATIONS FOR USE

ACL, MCL, LCL, PCL, rotational and combined instabilities

FEATURES AND BENEFITS

Anatomically correct Accutrac™ hinges with extension stops

Proprietary Sensil® padding eliminates migration

Flexible lower medial cuff provides increased protection against rotational forces and is breathable for added comfort

Skin-friendly SensEdge overmoulding on D-rings and lower cuff improves comfort and eliminates pressure points

Total Support System™ for ACL, MCL, LCL, PCL, rotational and combined instabilities

Internal D-rings for a more streamlined, lower profile fit

PCL version available

Hand-laminated, pre-preg carbon fibre, rigid frame

Corrosion resistant materials, ideal for water sports

Pro Sport model for the highest impact levels

Standard model for moderate to high impact levels

Vapor model for low to moderate impact levels

Custom paint options available

REQUIRED MEASUREMENTS

Use the Digital Measuring System below. More information is available on www.ossur.co.uk/DMS



Standard



PCL version

Ordering Information

CTi® Custom

Standard		Pro Sport		Vapor	
Left	Right	Left	Right	Left	Right
B-116600010	B-116500010	B-117600010	B-117500010	B-118600010	B-118500010

Digital Measuring System (DMS)

This innovative system requires minimal physical measurements and two digital photos, which can then be uploaded directly onto the Össur website (www.ossur.co.uk/dms) or sent to Customer Care (ossuruk@ossur.com). This new system is a quick, easy and reliable method of measuring for a CTi Custom without the need for training. Full instructions are available online and the DMS kit is available free of charge through Customer Care, tel: 08450 065 065. Sample measurement chart on page 137.



Description	Item No.
Digital Measuring System*	89100

*Digital camera required

CTi® Custom

Accessories/options

Description	Item No.
Replacement Condyle Pad Kit	B-704130050
Cotton Condyle Pad Covers (Pair)	13535
Extension Stops Kit	22130
Flexion Stop Kit (Requires assembly)	19254
Patella Cup	B-711130000
Gear Guards	22101

Contact Customer Care on 08450 065 065 to order AMS wraps, strap kits or replacement tibial silicone liners

Kits

Size	Moto X Kit*
Small	B-705130072
Medium	B-705130073
Large	B-705130074

*Moto X kit includes patella cup, gear guards, under sleeve and CTi Stickers

Sofsleeves

Size	Circumference	Coolmax Sleeve White	Sofsleeve	Undersleeve	Oversleeve	Sport Sleeve	Sport Padded Oversleeve
Small	< 38 cm	B-250031802	20643	20251	20051	20682	
Medium	38 - 45 cm	B-250031803	20644	20252	20052	20683	20448
Large	45 - 49.5 cm	B-250031804	20644	20253	20053	20684	20449
X-Large	49.5 - 54.5 cm		20645	20254	20054	20685	
XX-Large	54.5 - 62 cm		20645	20255	20055		
XXX-Large	62 - 71 cm		20646	20256	20056		
XXXX-Large	71 - 86 cm		20647	20257	20057		

Calf Pad Silicone CTi® Custom - Black

Size	Measurements	Item No.
Small	9 cm	B-304010057
M/Large	12.5 cm	B-304010058
X/XX Large	17.5 cm	B-304010059

CTi Custom 12 standard colours



CTi Custom Paint is also available for an additional fee (examples below). Contact Customer Care on 08450 065 065 for more information



CTi® OTS

The brace that has set the standard for truly custom ligament knee bracing for over 25 years is available off-the-shelf. Substituting elements of the rigid CTi® frame with a flexible subshell and adjustable tibial strap, CTi® OTS provides adjustability and an excellent out of the box fit while maintaining the level of support and protection that you expect from a CTi® knee brace.



INDICATIONS FOR USE

ACL, MCL, LCL, PCL, rotational and combined instabilities

FEATURES AND BENEFITS

Anatomically correct Accutrac™ hinges with extension stops

Flexible subshell and adjustable tibial strap capture the tibia and allow adjustments in hinge depth for a more customised fit

Proprietary Sensil® padding eliminates migration

Skin-friendly SensEdge overmoulding on buckles and lower cuff improves comfort and eliminates pressure points

Total Support System™ for ACL, MCL, LCL, PCL, rotational and combined instabilities

Internal D-rings for a streamlined, low profile fit

Hand-laminated, pre-preg carbon fibre, rigid frame

Corrosion resistant materials, ideal for water sports

Standard model for moderate to high impact levels

Pro Sport model for the highest impact levels

REQUIRED MEASUREMENTS

Caliper measurement: medio-lateral at joint width in an extended, weight bearing position

Circumference measurement: at the distal border of the patella



Ordering Information

Standard model - for moderate to high impact levels

Pro Sport model - a stronger brace for the highest impact levels

CTi® OTS			Standard		Pro Sport	
Size	Caliper	Circumference	Left	Right	Left	Right
Small	8.9 - 10.2 cm	27.9 - 32.1 cm	B-238600112	B-238500112	B-239600112	B-239500112
Medium	10.2 - 11.4 cm	32.1 - 36.2 cm	B-238600113	B-238500113	B-239600113	B-239500113
Large	11.4 - 12.4 cm	36.2 - 39.1 cm	B-238600114	B-238500114	B-239600114	B-239500114
X-Large	12.4 - 13.3 cm	39.1 - 42.2 cm	B-238600115	B-238500115	B-239600115	B-239500115
XX-Large	13.3 - 14.6 cm	42.2 - 46 cm	B-238600116	B-238500116	B-239600116	B-239500116

CTi® OTS PCL			Standard		Pro Sport	
Size	Caliper	Circumference	Left	Right	Left	Right
Small	8.9 - 10.2 cm	27.9 - 32.1 cm	B-238640112	B-238540112	B-239640112	B-239540112
Medium	10.2 - 11.4 cm	32.1 - 36.2 cm	B-238640113	B-238540113	B-239640113	B-239540113
Large	11.4 - 12.4 cm	36.2 - 39.1 cm	B-238640114	B-238540114	B-239640114	B-239540114
X-Large	12.4 - 13.3 cm	39.1 - 42.2 cm	B-238640115	B-238540115	B-239640115	B-239540115
XX-Large	13.3 - 14.6 cm	42.2 - 46 cm	B-238640116	B-238540116	B-239640116	B-239540116

Accessories/options

Moto X Kit

Size	Item No.
Small	B-705130072
Medium	B-705130073
Large	B-705130074

**Moto X kit includes patella cup, gear guards, under sleeve and CTi Stickers



Motocross Kit

Description	Item No.
Replacement Condyle Pad Kit	B-704130050
Cotton Condyle Pad Covers (pair)	13535
Extension Stop Kit	22130
Flexion Stop Kit (requires assembly)	19254
Patella Cup	B-711130000
Gear Guards	22101
Strap Kit* - Small	B-701140002
Strap Kit* - Medium	B-701140003
Strap Kit* - Large	B-701140004
Strap Kit* - X-Large/XX-Large	B-701140005

*Additional required strap for PCL brace, contact Customer Care on 08450 065 065

Anti Migration System Wrap

Size	Item No.
AMS Wrap - Small/Medium	B-410000001
AMS Wrap - Large/X-Large/XX-Large	B-410000002

**Silicone Liner CTi® OTS - Tibial Section**

Size	Item No.
Small	B-410000112
Medium	B-410000113
Large	B-410000114
X-Large	B-410000115
XX-Large	B-410000116

Silicone Liner CTi® OTS - Thigh Section

Size	Item No.
Small/Medium	B-410000122
Large/X-Large/XX-Large	B-410000124

Silicone Calf Pad CTi® OTS

Size	Measurements	Item No.
Small	9 cm	B-304010057
M/Large	12.5 cm	B-304010058
X-Large/XX-Large	17.5 cm	B-304010059

Flex™



INDICATIONS FOR USE

ACL, MCL, LCL, PCL, rotational and combined instabilities
 Long-term use for post operative, rehab and functional support
 Recommended for low to moderate activity

FEATURES AND BENEFITS

Accutrac™ ROM hinges with easy to use flexion/extension stops
 Heat-mouldable, carbon composite frame accommodates oedema or changes in leg shape during rehabilitation
 Gel Fit™ Condylar pad for improved suspension and comfort
 Total Support System™ for ACL, MCL, LCL, PCL, rotational and combined instabilities
 Adjustable tibial strap captures the tibia and allows adjustments in hinge depth for a customised fit
 Large, 'open window' frame design accommodates post op dressing and oedema
 Corrosion resistant materials, ideal for water sports

REQUIRED MEASUREMENTS

Caliper measurement: medio-laterally at joint width in a standard, weight-bearing position
 Circumference measurement: at the distal border of the patella

Ordering Information

Flex™		Standard		PCL		
Size	Caliper	Circ.	Left	Right	Left	Right
Small	8.9 - 10.2 cm	27.9 - 32.1 cm	FLEX-SL	FLEX-SR	FLEXPCL-SL	FLEXPCL-SR
Medium	10.2 - 11.4 cm	32.1 - 36.2 cm	FLEX-ML	FLEX-MR	FLEXPCL-ML	FLEXPCL-MR
Large	11.4 - 12.4 cm	36.2 - 39.1 cm	FLEX-LL	FLEX-LR	FLEXPCL-LL	FLEXPCL-LR
X-Large	12.4 - 13.3 cm	39.1 - 42.2 cm	FLEX-XLL	FLEX-XLR	FLEXPCL-XLL	FLEXPCL-XLR

Paradigm® OTS



INDICATIONS FOR USE

ACL, MCL, LCL, rotational and combined instabilities

Protection and stabilisation of surgical grafts

Recommended for low to moderate activity

FEATURES AND BENEFITS

Carbon fibre TriaX® shells with flexible subshell provides great support for low to moderate activity

Hard-anodised, polycentric hinges

Adjustable tibial strap captures the tibia and allows adjustments in hinge depth for a more customised fit

ROM kit included, enabling post surgical use

Large, 'open window' frame design for comfort and compliance

REQUIRED MEASUREMENTS

Circumference measurement: 15 cm above mid-patella

Ordering Information

Paradigm® OTS		Standard		PCL	
Size	Circ.	Left	Right	Left	Right
X-Small	34.3 - 39.4 cm	B-210600001	B-210500001	B-211600001	B-211500001
Small	39.4 - 44.5 cm	B-210600002	B-210500002	B-211600002	B-211500002
Medium	44.5 - 52.1 cm	B-210600003	B-210500003	B-211600003	B-211500003
Large	52.1 - 59.7 cm	B-210600004	B-210500004	B-211600004	B-211500004
X-Large	59.7 - 67.3 cm	B-210600005	B-210500005	B-211600005	B-211500005
XX-Large	67.3 - 74.9 cm	B-210600006	B-210500006	B-211600006	B-211500006

Accessories/options

Size	Strap and Pad	Liner Set Right	Liner Set Left
X-Small	B-701060001	B-703065001	B-703066001
Small	B-701060002	B-703065002	B-703066002
Medium	B-701060003	B-703065003	B-703066003
Large	B-701060004	B-703065004	B-703066004
X-Large	B-701060005	B-703065005	B-703066005
XX-Large	B-701060006	B-703065006	B-703066006

Description	Item No.
Flexion / Extension Stop Kit	B-705000504

Trainer™



INDICATIONS FOR USE

Mild to moderate ligament laxity and acute management of Grade I and Grade II MCL sprains

Recommended for low to moderate activity

FEATURES AND BENEFITS

Extremely durable, injection moulded TriaX® shells combine strength with flexibility

Unique Dynamic Force Strap reduces the load on the healing MCL

Tempered carbon steel ROM hinge with flexion and extension stops

Adjustable tibial strap for improved suspension and fit

Dynamic Force Strap tightens with extension to resist valgus forces and minimise axial rotation

REQUIRED MEASUREMENTS

Circumference measurement: 15 cm above mid-patella

Ordering Information

Trainer™

Size	Circumference	Left	Right
X-Small	34.3 - 39.4 cm	B-237600001	B-237500001
Small	39.4 - 44.5 cm	B-237600002	B-237500002
Medium	44.5 - 52.1 cm	B-237600003	B-237500003
Large	52.1 - 59.7 cm	B-237600004	B-237500004
X-Large	59.7 - 67.3 cm	B-237600005	B-237500005

Knee OA

Unloader One® Custom	102
Unloader One® OTS.....	104
CTi® OA	107
OAsys® Carticare	109

Unloader One® Custom

A market leader in the field of osteoarthritis knee bracing for many years, the Unloader One® has been redesigned to provide numerous benefits to both the clinician and patient, specifically improved ease of use, improved comfort and a more intuitive fitting / servicing design.

As ever, the brace combines a streamlined, flexible single upright with Dual Dynamic Force Straps to generate a highly effective unloading force for unicompartmental conditions. A ratcheting Dynamic Force Control System empowers the patient to manage the therapeutic effects of the brace whilst liners coated with Össur Sensil® Silicone Technology eliminate migration.



INDICATIONS FOR USE

Mild to severe unicompartmental osteoarthritis

Osteoarthritic knee pain

Unicompartmental knee conditions requiring load reduction, such as articular cartilage defect repair, meniscal cartilage repair, avascular necrosis

Suitable for all activity levels

FEATURES AND BENEFITS

Dual Dynamic Force Straps increase unloading capabilities and prevent rotation of the brace on the leg

Tool-free, ratcheting Dynamic Force Control System enables the patient to manage their own pain

Quick fit buckles enable easy application and removal without the need for cumbersome strap adjustment

Breathable liners featuring Össur Sensil® Silicone Technology on an Aerospacer base material eliminate migration

Flexible, low profile upright contours to patient anatomy and allows use underneath clothing


Also available in Plus model, equipped with broader shells, wider thigh strap plus a gastroc strap that helps secure the brace in place. Including a sturdier upright with Adjustable Dynamic joint for additional leverage

Quick Connect Strapping for tool-free removal and replacement

Lightweight frame is perforated for flexibility and ventilation

REQUIRED MEASUREMENTS

Use the Digital Measuring System overleaf. More information is available on www.ossur.co.uk/DMS

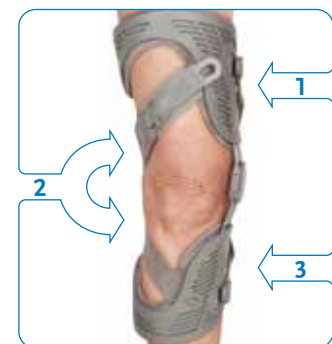

Force Diagram

Unloader One® Custom

Ordering Information

Unloader One® Custom

Colour	Left	Right
Black	B-115600111	B-115500111
Grey	B-115601711	B-115501711

To order an Unloader One® Custom, please complete the order form at the end of this section and take a negative cast, or use the Digital Measuring System as detailed below

Unloader One Plus Custom

Colour	Thigh circumference	Calf circumference	Left	Right
Black	45.7 - 83.8cm	35.6 - 61cm	B-119600100	B-119500100
Grey	45.7 - 83.8cm	35.6 - 61cm	B-119601700	B-119501700

Digital Measuring System (DMS)

This innovative system requires minimal physical measurements and two digital photos, which can then be uploaded directly onto the Össur website (www.ossur.co.uk/dms) or sent to Customer Care (ossuruk@ossur.com). This new system is a quick, easy and reliable method of measuring for an Unloader One Custom without the need for training. Full instructions are available online and the DMS kit is available free of charge through Customer Care, tel: 08450 065 065. Sample measurement chart on page 137.

Description	Item No.
Digital Measuring System	89100



Accessories/options

Silicone AMS (Anti-Migration System) Wrap

Right lateral / Left medial

Right medial / Left lateral

Size	Circ.	Grey	Black	Grey	Black
X-Small	< 25 cm	B-715218031	B-715218011	B-715217031	B-715217011
Small	25 - 35.5 cm	B-715218032	B-715218012	B-715217032	B-715217012
Medium	35.5 - 42.5 cm	B-715218033	B-715218013	B-715217033	B-715217013
Large	42.5 - 51 cm	B-715218034	B-715218014	B-715217034	B-715217014
X-Large	51 - 61.5 cm	B-715218035	B-715218015	B-715217035	B-715217015

AMS Wraps are for high activity users or to improve suspension of the brace

Accessory Kits

Grey

Black

Description	RM/LL	RL/LM	RM/LL	RL/LM
Strap & Strap Pad & Buckle Clasp Kit	B-701217008	B-701218008	B-701217009	B-701218009
Strap & Strap Pad Kit	B-701210006		B-701210007	
Strap Kit	B-701210004		B-701210005	
Pad Kit	B-702210003		B-702210001	
Buckle Clasp Kit	B-708217003	B-708218003	B-708217001	B-708218001
Alligator Clip Kit (included in all strap kits)	B-704210003		B-704210001	
Extension Stop Kit	B-301040046		B-301040047	

To order replacement liners please contact Customer Care on 08450 065 065

Unloader One Custom 12 Standard colours



WEB www.ossur.co.uk

E-MAIL ossuruk@ossur.com

TEL 08450 065 065

FAX 0161 490 8501

Unloader One® OTS

A market leader in the field of osteoarthritis knee bracing for many years, the Unloader One has been redesigned to provide numerous benefits to both the clinician and patient, specifically improved ease of use, improved comfort and a more intuitive fitting / servicing.

The brace combines a streamlined, flexible single upright with Dual Dynamic Force Straps to generate a highly effective unloading force for unicompartmental conditions. A ratcheting Dynamic Force Control System empowers the patient to manage the therapeutic effects of the brace whilst liners coated with Össur Sensil® Silicone Technology eliminate migration. The flexible perforated shells conform easily to patient anatomy, giving Unloader One OTS an excellent out of the box fit. We have a broad range of sizes including the Unloader One OTS Short and Unloader One Plus for larger patients.



INDICATIONS FOR USE

Mild to severe unicompartmental osteoarthritis

Osteoarthritic knee pain

Unicompartmental knee conditions requiring load reduction, such as articular cartilage defect repair, meniscal cartilage repair, avascular necrosis, or tibia plateau fracture

Suitable for all activity levels

FEATURES AND BENEFITS

Dual Dynamic Force Straps increase unloading capabilities and eliminate rotational tendencies caused by a single strap

Tool free, ratcheting Dynamic Force Control System enables the patient to manage their own pain

Quick fit buckles enable easy application and removal without the need for cumbersome strap adjustment

An Aerospacer base material coated with Össur Sensil® Silicone Technology results in a breathable padding that won't migrate

Flexible, low profile upright contours to patient anatomy and allows use underneath clothing

Quick Connect Strapping for tool-free removal and replacement

Also available in Plus model, equipped with broader shells, wider thigh strap plus a gastroc strap that helps secure the brace in place. Including a sturdier upright with Adjustable Dynamic joint for additional leverage.

Lightweight frame is perforated for flexibility and ventilation

REQUIRED MEASUREMENTS


Circumference measurement: 15 cm below mid-patella



Unloader One® OTS

Ordering Information

Unloader One® OTS		Hinge medial*	
Size	Circ.	Left	Right
Small	25 - 35.5 cm	B-240619712	B-240519712
Medium	35.5 - 42.5 cm	B-240619713	B-240519713
Large	42.5 - 51 cm	B-240619714	B-240519714
X-Large	51 - 61.5 cm	B-240619715	B-240519715

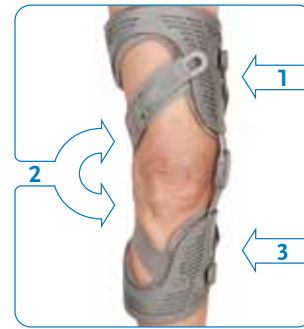

On a right leg, with the medial side affected, the hinge goes on the medial side of the knee

Unloader One® OTS		Hinge lateral*	
Size	Circ.	Left	Right
Small	25 - 35.5 cm	B-240629712	B-240529712
Medium	35.5 - 42.5 cm	B-240629713	B-240529713
Large	42.5 - 51 cm	B-240629714	B-240529714
X-Large	51 - 61.5 cm	B-240629715	B-240529715

On a left leg, with the lateral side affected, the hinge goes on the lateral side of the knee

* Hinge is placed on the side of the affected compartment

Unloader One Short OTS		Hinge medial*		Hinge lateral*	
Size	Circumference	Left	Right	Left	Right
X-Small	25 - 31cm	B-241619711	B-241519711	B-241629711	B-241529711
Small	31 - 35.5cm	B-241619712	B-241519712	B-241629712	B-241529712
Medium	35.5 - 42.5cm	B-241619713	B-241519713	B-241629713	B-241529713

* Hinge is placed on the side of the affected compartment

Unloader One Plus OTS, standard version			Hinge medial*		Hinge lateral*	
Size	Circumference	Calf circumference	Left	Right	Left	Right
Large	53 - 64cm	36 - 43cm	B-245610004	B-245510004	B-245620004	B-245520004
X-Large	64 - 74cm	43 - 51cm	B-245610005	B-245510005	B-245620005	B-245520005

Standard version is recommended for patients less than 6'3" tall.

Unloader One Plus OTS, long version			Hinge medial*		Hinge lateral*	
Size	Circumference	Calf circumference	Left	Right	Left	Right
Large	53 - 64cm	36 - 43cm	B-244610004	B-245510004	B-244620004	B-244520004
X-Large	64 - 74cm	43 - 51cm	B-244610005	B-245510005	B-244620005	B-244520005

Long version is recommended for patients taller than 6'3"



Unloader One® OTS

Accessories/options

Unloader One® AMS (Anti-Migration System) Wrap

Size	Circ.	Left Lateral / Right Medial	Right Lateral / Left Medial
X-Small	< 31 cm	B-715217031	B-715218031
Small	31 - 35.5 cm	B-715217032	B-715218032
Medium	35.5 - 42.5 cm	B-715217033	B-715218033
Large	42.5 - 51 cm	B-715217034	B-715218034
X-Large	51 - 61.5 cm	B-715217035	B-715218035

AMS Wraps are for high activity users or to improve suspension of the brace.

Unloader One® Silicone Calf Liners

Size	Left Lateral / Right Medial	Right Lateral / Left Medial
X-Small	B-419700311	B-419800311
Small	B-419700312	B-419800312
Medium	B-419700313	B-419800313
Large	B-419700314	B-419800314
X-Large	B-419700315	B-419800315

Unloader One® Silicone Thigh Liners

Size	Left Lateral / Right Medial	Right Lateral / Left Medial
X-Small	B-409700321	B-409800321
Small	B-409700322	B-409800322
Medium	B-409700323	B-409800323
Large	B-409700324	B-409800324
X-Large	B-409700325	B-409800325

Accessory Kits

Description	Left Lateral / Right Medial	Right Lateral / Left Medial
Strap & Strap Pad & Buckle Clasp Kit	B-701217008	B-701218008
Strap & Strap Pad Kit		B-701210006
Strap Kit		B-701210004
Pad Kit	B-702210003	
Quick Fit Buckle Kit	B-708217003	B-708218003
Alligator Clip Kit (included in all strap kits)		B-704210003
Flexion and Extension Stop Kit		B-301040046

CTi® OA

For over 25 years CTi® has set the standard for custom ligament bracing. Add varus or valgus correction to this concept and you have an excellent, dual hinged hybrid OA and ligament knee brace for your more active patients. New improvements include proprietary Sensil® padding that eliminates migration, a flexible lower medial calf cuff for greater protection against rotational forces and skin friendly SensEdge overmoulding on injection moulded parts to avoid pressure points.



INDICATIONS FOR USE

Mild to severe unicompartmental osteoarthritis
 ACL, MCL, LCL, PCL, rotational and combined instabilities
 Demanding activities of daily living

FEATURES AND BENEFITS

Anatomically correct Accutrac™ hinges with extension stops
 Proprietary Sensil® padding eliminates migration
 Flexible lower medial calf cuff provides increased protection against rotational forces and is breathable for added comfort
 Skin-friendly SensEdge overmoulding on buckles and lower cuff improves comfort and eliminates pressure points
 Total Support System™ for ACL, MCL, LCL, PCL, rotational and combined instabilities
 Hand-laminated, pre-preg carbon fibre, rigid frame construction
 Custom-fabricated to each patient's unique measurements with up to 7° varus/valgus correction built into the frame
 Medial arm available in soft or rigid material (one included)
 Internal D-rings for a more streamlined, lower profile fit
 Corrosion resistant materials, ideal for water sports

REQUIRED MEASUREMENTS

Use the Digital Measuring System overleaf. More information is available on www.ossur.co.uk/DMS



CTi® OA

Ordering Information

CTi® OA

Description	Left	Right
CTi® OA	B-116630010	B-116530010

Digital Measuring System (DMS)

This innovative system requires minimal physical measurements and two digital photos, which can then be uploaded directly onto the Össur website (www.ossur.co.uk/dms) or sent to Customer Care (ossuruk@ossur.com). This new system is a quick, easy and reliable method of measuring for a Custom CTi OA without the need for training. Full instructions are available online and the DMS kit is available free of charge through Customer Care, tel: 08450 065 065. Sample measurement chart on page 137.



Description	Item No.
Digital Measuring System	89100

OAsys® Carticare



INDICATIONS FOR USE

Medial or lateral compartment unloading for unicompartmental OA or articular cartilage healing

Post operative, rehab and functional support plus unloading of the medial or lateral compartment

Functional support for patients with varus or valgus alignment

Low to moderate activity level and daily living

FEATURES AND BENEFITS

Accutrac™ ROM hinges with easy to use flexion and extension stops

Acculign™ slide bar for on the spot medial or lateral unloading

Heat-mouldable frame for customised fitting

Total Support System™ for ACL, MCL, LCL, rotational and combined instabilities

Exclusive Gel Fit™ condylar pads provide joint line comfort and cushion bony prominences

Corrosion resistant materials, ideal for water sports

REQUIRED MEASUREMENTS

Caliper measurement: medio-laterally at joint width in standard, weight-bearing position

Circumference measurement: at distal border of the patella

Ordering Information

OAsys® Carticare

Size	Caliper	Circ.	Left	Right
Small	8.6 - 9.8 cm	27.9 - 31.1 cm	OA3SL	OA3SR
Medium	9.8 - 11.1 cm	31.1 - 36.2 cm	OA3ML	OA3MR
Large	11.1 - 12.7 cm	36.2 - 40.3 cm	OA3LL	OA3LR
X-Large	12.7 - 13.9 cm	40.3 - 44.1 cm	OA3XLL	OA3XLR

Contact Customer Care on 08450 065 065 for spare straps & pads

Post-Op Knees

Innovator DLX®	111
Innovator®	112
Advance® Knee	113

Innovator DLX®



Cool version



Full foam version

INDICATIONS FOR USE

Post operative or post injury ROM control or immobilisation

FEATURES AND BENEFITS

Quick release buckles added for easy donning and doffing

One touch™ easy-to-adjust ROM hinge

Reusable ROM locks included

One touch™ drop lock feature can be locked at 0, 15, 30 and 45 degrees

Hyperextension stop at -10 degrees

Available in Cool and Full foam version. Can be worn over clothing or against the skin

One touch™ telescopic struts

Gastrocnemius strap eliminates migration

REQUIRED MEASUREMENTS

Thigh circumference: 15 cm above mid-patella

Ordering Information

Innovator DLX®

Size	Thigh circumference	Cool	Full foam
Regular	up to 69 cm	423100	423200
X-Large	69 - 89 cm	423101	423201

Add an "I" at the end of the item number to order the product in a bag only, without the box

Accessories/options

Description	Item No.
ROM lock kit	B-705180000

Replacement Softgoods

Size	Cool		Full Foam	
	Foam	Straps	Foam	Straps
Regular	B-703180001	B-701180001	B-703180021	B-701180021
X-Large	B-703180002	B-701180002		B-701180022



One touch™ easy-to-adjust ROM hinge



Quick release buckles



One touch™ telescopic struts



One touch™ drop lock

Innovator®



Cool version



Full foam version

INDICATIONS FOR USE

Post operative or post injury range of motion control or immobilisation

FEATURES AND BENEFITS

One touch™ easy to adjust ROM hinge

Flexion range can be quickly adjusted to 0, 10, 20, 30, 45, 60, 75, 90, 105 and 120°. Extension can be adjusted to 0, 10, 20, 30, 45, 60, 75 and 90°

Available in 2 versions: Full Foam or Cool

Standard version available in 3 lengths, and as full foam or cool version

REQUIRED MEASUREMENTS


Thigh circumference: 15 cm above the mid-patella.



One touch™ easy-to-adjust ROM hinge



Flexion settings 0 - 120°
Extension settings 0 - 90°

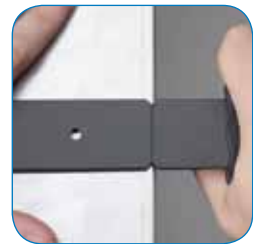

Snap off struts for easy shortening

Ordering Information

Innovator®		Full Foam			Cool	
Size	Thigh circ.	40 cm	61 cm	71 cm	61 cm	71 cm
Regular	up to 69 cm	403300	403200	403201	403100	403101
X-Large	69 - 89 cm	403301	403202	403203	403102	403103

Accessories/options

Description	Item No.
Cool length reduction kit	403150
Strap length reduction kit, Full Foam	403299
Strap length reduction kit, Cool	403199

Replacement Softgoods

Size	Full Foam	Cool
Regular	403275	403175
Regular Long	403276	
X-Large	403277	403176
X-Large Long	403278	

Advance® Knee



INDICATIONS FOR USE

Lost or limited range of motion

Post surgical contractures

Contractures resulting from burns or extended immobilisation

Non traumatic tissue remodelling

To achieve long term range of motion correction, without constant clinical supervision

CVA

FEATURES AND BENEFITS

Eccentric torsion spring can be adjusted to increase or decrease tension

Simple lever allows the spring to be locked safely during fitting and removal

Telescopic adjustable arms eliminate the need for different sizes

Malleable cuffs allow custom fitting

Cuffs lined with 'memory foam' pads help reduce pressure points

Available in Left and Right, Flexion and Extension models

REQUIRED MEASUREMENTS

One size fits all (Telescopic adjustment)

Ordering Information

Advance® Knee

Size	Left	Right
Extension *	199089	199090
Flexion **	199091	199092

* Order extension model for flexion contracture (applies an extension force)

** Order flexion model for extension contracture (applies a flexion force)

Accessories/options

Description	Item No.
Advance® Knee Extension Refurb Kit	199132
Advance® Knee Flexion Refurb Kit	199133

Knee Immobiliser

Exoform® Knee Immobiliser	115
Universal 3-Panel Knee Immobiliser.....	116
DeLuxe Knee Immobiliser	117

Exoform® Knee Immobiliser



Cool version



Full foam version

INDICATIONS FOR USE

Post injury	Post trauma
-------------	-------------

FEATURES AND BENEFITS

One size fits all

Slide-to-size adjustment enables you to provide the correct fit for a wide variety of patient lengths and widths

Popliteal pad provides superior fit and limits migration

Cool version can be worn directly against the skin for greater comfort

Full foam version provides warmth and compression

REQUIRED MEASUREMENTS

Universal - fits leg lengths of 46 - 61 cm

Ordering Information

Exoform Knee Immobiliser

Description	Leg circumference	Item No
Cool	up to 71.1cm	222100
Full foam	up to 71.1cm	222000
Full foam XXXL	up to 96.5cm	222050

Accessories/options

Description	Item No.
Replacement softgoods	222001

Universal 3-Panel Knee Immobiliser



INDICATIONS FOR USE

Post injury

Post trauma

FEATURES AND BENEFITS

Lightweight rigid posterior stays with adjustable medial and lateral stays

Available with optional hot and cold therapy gel pad

REQUIRED MEASUREMENTS

Universal, length of brace

Ordering Information

Length	Item No
36 cm	200010
41 cm	200016
51 cm	200020
56 cm	200022
61 cm	200030

Accessories/options

Description	Item No
Hot/cold therapy gel pad	GB7

DeLuxe Knee Immobiliser



INDICATIONS FOR USE

Post injury

Post trauma

FEATURES AND BENEFITS

Stays may be contoured for custom fit

3-panel construction for optimal fit and comfort

Simple touch and close fastening

REQUIRED MEASUREMENTS

Length of splint

Ordering Information

DeLuxe Knee Immobiliser

Length of splint	Item No.
30 cm	0312
35.6 cm	0314
40 cm	0316
45.7 cm	0318
50 cm	0320
55.8 cm	0322
60 cm	0324

Knee Supports

Bio Skin® Gladiator	119
Bio Skin® Hinged Knee Skin™	120
Bio Skin® Standard Knee Skin™	121
Form Fit® Knee Brace	122
ThermoWrap™	123

Bio Skin® Gladiator



Front closure (DT)



Pull-on (XT)

INDICATIONS FOR USE

Mild sprains and instabilities
 Inflammation (joint swelling)
 Medial Collateral Ligament (MCL) sprains
 Lateral Collateral Ligament (LCL) sprains
 Meniscal tears
 Osteoarthritis

FEATURES AND BENEFITS

Universal left or right
 Open patella
 8 strap system for greater compression and support
 Finger loops for fitting assist on pull-on version
 Bi-centric hinges with adjustable flexion and extension stop settings
 Extension limits between 0 - 90°
 Flexion limits between 0 - 90°
 Easy to adjust pins to lock hinges
 Contoured condyle pads
 38 cm in length
 Available in pull-on or front closure version

REQUIRED MEASUREMENTS

Knee circumference at mid-patella

Ordering Information

Bio Skin® Gladiator

Size	Knee circ.	Front closure (DT)	Pull-on (XT)
X-Small	31 - 33 cm	51360	41340
Small	33 - 36 cm	51361	41341
Medium	36 - 41 cm	51362	41342
Large	41 - 43 cm	51363	41343
X-Large	43 - 46 cm	51364	41344
XX-Large	46 - 54 cm	51365	41345

Bio Skin® Hinged Knee Skin™



Standard



With Front Closure

INDICATIONS FOR USE

- Mild sprains
- Inflammation (joint swelling)
- Medial Collateral Ligament (MCL) sprains
- Lateral Collateral Ligament (LCL) sprains
- Meniscal tears
- Early osteoarthritis
- Patella tendonitis

FEATURES AND BENEFITS

- Universal, left or right
- Open patella
- 4 strap system for greater compression
- Donut buttress for extra patella support
- Finger loops to assist application
- Bi-centric hinged aluminium side supports for medial/lateral support
- Available in pull-on or front closure version
- Available in three types of Bio Skin® material (See page 6/7)

REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

Ordering Information

Bio Skin® Hinged Knee Skin™		Ultima	Ultima 2s	Ultima 2sL
Size	Knee circ.	Pull-on	Pull-on	Front closure
X-Small	31 - 33 cm	41300-O	51300O	
Small	33 - 36 cm	41301-O	51301O	51331
Medium	36 - 41 cm	41302-O	51302O	51332
Large	41 - 43 cm	41303-O	51303O	51333
X-Large	43 - 46 cm	41304-O	51304O	51334
XX-Large	46 - 54 cm	41305-O	51305O	51335

Bio Skin® Standard Knee Skin™



INDICATIONS FOR USE

Mild sprains and strains
 Inflammation (joint swelling)
 Meniscal tears

FEATURES AND BENEFITS

Universal, left or right
 Open patella
 Available with or without straps
 Available in two types of Bio Skin® material (See page 6/7)

REQUIRED MEASUREMENTS

Knee circumference at mid-patella

Ordering Information

Bio Skin® Standard Knee Skin™

Size	Knee circ.	Ultima		Ultima 2s
		w. Straps	w/o Straps	w/o Straps
X-Small	31 - 33 cm	41110	41100	51100
Small	33 - 36 cm	41111	41101	51101
Medium	36 - 41 cm	41112	41102	51102
Large	41 - 43 cm	41113	41103	51103
X-Large	43 - 46 cm	41114	41104	51104
XX-Large	46 - 54 cm	41115	41105	51105

Form Fit[®] Knee Brace



INDICATIONS FOR USE

Mild to moderate sprains and strains

Mild to moderate Anterior Cruciate Ligament (ACL) and Posterior Cruciate Ligament (PCL) tears

Medial Collateral Ligament (MCL) and Lateral Collateral Ligament (LCL) tears

FEATURES AND BENEFITS

Accutrac ROM hinges provide anatomically-accurate tracking, extension control and additional medial-lateral stability

Open popliteal area reduces material bulging and irritation

Neoprene material provides comfort, retains heat for cold weather conditions and is highly durable for use in sports activities

Malleable rigid hinge arms provide intimate fit

Universal buttresses are included for customizable patella support

REQUIRED MEASUREMENTS

Circumference: 15 cm above mid-patella

Ordering Information

Form Fit ROM Knee Sleeve

Size	Circumference	Item No.
X-Small	33 - 39cm	508352
Small	39 - 47cm	508353
Medium	47 - 53cm	508355
Large	53 - 60cm	508357
X-Large	60 - 67cm	508358
XX-Large	67 - 75cm	508359

ThermoWrap™



Bulk Ordering Discounts

Order Quantity	Discount
10	15%
25	25%
50	35%

See page 2 for terms and conditions

INDICATIONS FOR USE

Mild sprains

Inflammation (joint swelling)

Medial Collateral Ligament (MCL) sprains

Lateral Collateral Ligament (LCL) sprains

Meniscal tears

Early osteoarthritis

Patella tendonitis

FEATURES AND BENEFITS

Provides increased medial/lateral stability, compression and therapeutic warmth

Enclosed stainless steel hyperextension control hinges

Hyperextension model includes additional strapping posteriorly to control hyperextension

REQUIRED MEASUREMENTS

Knee circumference at mid-patella

Ordering Information

Thermowrap™

Size	Knee circumference	Standard	Hyperextension
X-Small	25 - 30.5 cm	0292+	0292/HE
Small	27 - 35.5 cm	0293+	0293/HE
Medium	33 - 40.5 cm	0295+	0295/HE
Large	38 - 45.5 cm	0297+	0297/HE
X-Large	43 - 55.5 cm	0298+	0298/HE

Knee Patella

Bio Skin® Q Lok Brace.....	125
Bio Skin® Q Brace™	126
Bio Skin® Patella Stabiliser.....	127
Bio Skin® Visco Knee Skin™	128
Bio Skin® Q Baby™ Patella Tendon Strap	129
Neoprene Knee Support with Stabilised Patella.....	130

Bio Skin® Q Lok Brace



Pull-on



With Front Closure

INDICATIONS FOR USE

- Patello femoral pain
- Anterior knee pain
- Medial or lateral patella tracking
- Medial or lateral patella subluxation
- Chondromalacia patella

FEATURES AND BENEFITS

- Seats patella in trochlear notch
- Reduces patella tilt
- Provides a traction force on the lateral retinaculum
- Stretches lateral tissue
- Lightweight hinges anchor the Dynamic Patella Traction Strap™
- Contoured cut designed to fit a wider variety of thigh shapes
- Hinges naturally contour to leg
- Comfortable and breathable Stratus™ material
- Universal, left or right
- Available in pull-on or front closure

REQUIRED MEASUREMENTS

- Knee circumference at mid patella

Ordering Information

Bio Skin® Q Lok Brace

Size	Knee circumference	Pull-on	Front Closure
Small	33 - 36 cm	61641	51641
Medium	36 - 41 cm	61642	51642
Large	41 - 43 cm	61643	51643
X-Large	43 - 46 cm	61644	51644
XX-Large	46 - 54 cm	61645	51645

Bio Skin® Q Brace™



INDICATIONS FOR USE

- Anterior knee pain
- Medial or lateral patella tracking
- Medial or lateral patella subluxation
- Chondromalacia patella

FEATURES AND BENEFITS

- Adjustable 'T' strap for dynamic directional patella tracking control
- Universal, left or right
- Open patella
- 4 strap system for greater compression
- Available in pull-on or front closure versions
- Available in three types of Bio Skin® material (See page 6/7)

REQUIRED MEASUREMENTS

- Knee circumference at mid patella

Ordering Information

Bio Skin® Q Brace™		Ultima	Ultima 2s	Ultima 2sL
Size	Knee circ.	Pull-on	Pull-on	Front Closure
X-Small	31 - 33 cm	41610-O	51610	51630
Small	33 - 36 cm	41611-O	51611	51631
Medium	36 - 41 cm	41612-O	51612	51632
Large	41 - 43 cm	41613-O	51613	51633
X-Large	43 - 46 cm	41614-O	51614	51634
XX-Large	46 - 54 cm	41615-O	51615	51635

Bio Skin® Patella Stabiliser



INDICATIONS FOR USE

Chondromalacia patella

Inflammation

Anterior knee pain

Meniscal tears

Mild sprains

Patella maltracking

FEATURES AND BENEFITS

Universal, left or right

Open patella

4 strap system for greater compression

Donut buttresses for effective patella stabilisation

Available in two types of Bio Skin® material (See page 6/7)

REQUIRED MEASUREMENTS

Knee circumference at mid patella

Ordering Information

Bio Skin® Patella Stabiliser

Size	Knee circ.	Ultima	Ultima 2s
X-Small	31 - 33 cm	41250-O	51250O
Small	33 - 36 cm	41251-O	51251O
Medium	36 - 41 cm	41252-O	51252O
Large	41 - 43 cm	41253-O	51253O
X-Large	43 - 46 cm	41254-O	51254O
XX-Large	46 - 54 cm	41255-O	51255O

Bio Skin® Visco Knee Skin™



INDICATIONS FOR USE

- Mild sprains
- Inflammation
- Post arthroscopy
- Meniscal tears
- Patella tendonitis

FEATURES AND BENEFITS

- Universal, left or right
- Visco polymer patella ring for passive massage
- Finger loops to assist application
- Available in two types of Bio Skin® material (See page 6/7)
- Available with or without straps

REQUIRED MEASUREMENTS

- Knee circumference at mid patella

Ordering Information

Bio Skin® Visco Knee Skin™		Ultima		Ultima 2s	
Size	Knee circ.	w. Straps	w/o Straps	w. Straps	w/o Straps
X-Small	31 - 33 cm	41710	41700	51710	51700
Small	33 - 36 cm	41711	41701	51711	51701
Medium	36 - 41 cm	41712	41702	51712	51702
Large	41 - 43 cm	41713	41703	51713	51703
X-Large	43 - 46 cm	41714	41704	51714	51704
XX-Large	46 - 54 cm	41715	41705	51715	51705

Bio Skin® Q Baby™ Patella Tendon Strap



INDICATIONS FOR USE

Patella tendonitis
 Patella maltracking
 Osgood-Schlatter's disease

FEATURES AND BENEFITS

Easy to apply
 Uniquely shaped silicone pad fits over the patella tendon
 Fully adjustable velcro exterior for custom sizing
 Soft Bio Skin® Ultima 2sL micro fleece for comfort

REQUIRED MEASUREMENTS

Circumference at distal patella

Ordering Information

Bio Skin® Q Baby™ Patella Tendon Strap

Size	Distal patella circumference	Item No.
X-Small/Medium	31 - 41 cm	51652
Large/XX-Large	41 - 54 cm	51655

Neoprene Knee Support with Stabilised Patella



INDICATIONS FOR USE

- Osteoarthritis
- Chondromalacia patella
- Anterior knee pain
- Mild sprains
- Inflammation
- Meniscal tears
- Patella maltracking

FEATURES AND BENEFITS

- Moulded material to maintain a low profile
- Accordion moulding in the popliteal area eliminates bunching

REQUIRED MEASUREMENTS

- Circumference at mid patella

Ordering Information

Neoprene Knee Support With Stabilised Patella & Donut Buttress

Size	Knee circumference	Universal	Sewn In Donut	Sewn In U-Shaped
Small	29 - 33 cm	302563BLK	303563BLK	304563BLK
Medium	33 - 37 cm	302565BLK	303565BLK	304565BLK
Large	37 - 41 cm	302567BLK	303567BLK	304567BLK
X-Large	41 - 44 cm	302568BLK	303568BLK	304568BLK
XX-Large	44 - 48 cm	302569BLK	303569BLK	304569BLK

Bulk Ordering Discounts

Order Quantity	Discount
20	15%
30	25%
50	35%

See page 2 for terms and conditions

Sleeves

Bio Skin® Under Skin™	132
Bio Skin® Thigh Skin™	133
Bio Skin® Calf Skin™	134
Bio Skin® Shinsplints Wrap/Band	135

Bio Skin® Under Skin™



INDICATIONS FOR USE

For mild compression and increased proprioception

Can be worn under a functional knee brace

FEATURES AND BENEFITS

Holds functional knee brace in position

50 cm length

Made from Bio Skin® Ultima™ material

REQUIRED MEASUREMENTS

Circumference at mid patella

Circumference 18 cm above mid patella with knee fully extended

Ordering Information

Bio Skin® Under Skin™

Size	Mid patella	Circ. 18 cm above mid patella	Item No.
X-Small	31 - 33 cm	41 - 46 cm	41400
Small	33 - 36 cm	46 - 51 cm	41401
Medium	36 - 41 cm	51 - 56 cm	41402
Large	41 - 43 cm	56 - 64 cm	41403
X-Large	43 - 46 cm	64 - 71 cm	41404
XX-Large	46 - 54 cm	71 - 81 cm	41405

Bio Skin® Thigh Skin™



Standard



With cinch strap

INDICATIONS FOR USE

Quad bruising
 Iliofemoral sprain
 Thigh sprain

FEATURES AND BENEFITS

Breathable material releases excess heat and moisture
 Prevents build up of sweat
 Applies constant compression
 Increases proprioception
 Contoured fit
 Lightweight
 Made from Bio Skin® Ultima™ material (See page 6/7)
 Available with cinch strap to isolate compression

REQUIRED MEASUREMENTS

Mid thigh circumference

Ordering Information

Bio Skin® Thigh Skin™

Ultima

Size	Mid thigh circ.	w. Strap	w/o Strap
X-Small	41 - 46 cm	46110	46100
Small	46 - 51 cm	46111	46101
Medium	51 - 56 cm	46112	46102
Large	56 - 64 cm	46113	46103
X-Large	64 - 71 cm	46114	46104
XX-Large	71 - 81 cm	46115	46105

Bio Skin® Calf Skin™



INDICATIONS FOR USE

Bruising

Shin splints

Strains

FEATURES AND BENEFITS

Breathable material releases excess heat and moisture

Prevents build up of sweat

Applies constant compression

Increases proprioception

Contoured fit

Made from Bio Skin® Ultima™ material (See page 6/7)

Lightweight

REQUIRED MEASUREMENTS

Circumference at largest part of calf

Ordering Information

Bio Skin® Calf Skin™

Size	Calf circumference	Item No.
X-Small	20 - 25 cm	46300
Small	25 - 30 cm	46301
Medium	30 - 35 cm	46302
Large	35 - 41 cm	46303
X-Large	41 - 46 cm	46304
XX-Large	46 - 51 cm	46305

Bio Skin® Shinsplints Wrap/Band



INDICATIONS FOR USE

Bruising
 Shin splints
 Strains

FEATURES AND BENEFITS

Breathable material releases excess heat and moisture
 Prevents build up of sweat
 Applies constant compression
 Increases proprioception
 Contoured fit
 Lightweight
 Available in two lengths, 3" or 5" (8 cm or 13 cm)
 Wrap around for contoured fit
 Reduces muscle vibration

REQUIRED MEASUREMENTS

Circumference at largest part of calf

Ordering Information

Bio Skin® Shin Splints Band

Size	Calf circumference	8 cm width	13 cm width
X-Small - Medium	20 - 35 cm	56312	56322
Large - XX-Large	35 - 51 cm	56315	56325

Specials - Custom Thermowrap

Made-to-measure product

To place your order please photocopy this form, complete the details and fax to Össur® 0161 490 8501

Order Information

Order No _____

Preferred delivery date _____

Date of order _____

Hospital name _____

Product _____

Options _____

Colour _____


Material _____

Size _____

Left / Right _____

Options _____

Measurements

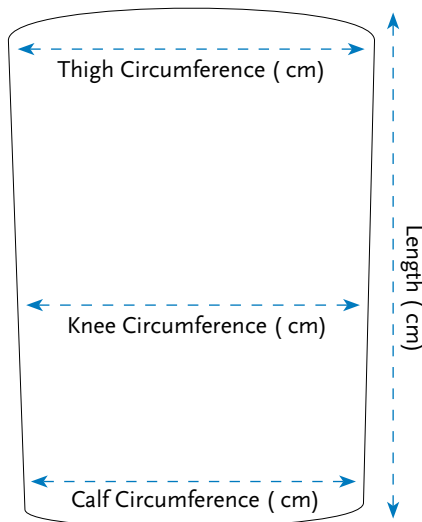

Thigh _____ cm

Knee _____ cm

Calf _____ cm

Length _____ cm

Other info/Instructions _____

CTi® Custom / Unloader One® Custom

ORDER FORM FOR USE WITH DMS KIT

Össur UK, Building 3000, Manchester Business Park, Aviator Way, Manchester, M22 5TG
 Tel: +44 (0) 8450 065 065 Fax: +44 (0) 161 490 8501 Email: ossuruk@ossur.com

Patient Information


Name :			
Date of Birth :	Contact No.		
Height :	Weight		
Diagnosis :			

Measurements

M-L Caliper	Left	Right
Circumferences		
15 cm Above mid patella :		
7.5 cm Above mid patella :		
7.5 cm Below mid patella :		
15 cm Below mid patella :		

Brace Details

Left / Right / Bilateral :			
CTi :	Pro Sport <input type="checkbox"/>	Standard <input type="checkbox"/>	Vapor <input type="checkbox"/>
	MX Kit <input type="checkbox"/>	PCL <input type="checkbox"/>	
Unloader :	Medial <input type="checkbox"/>	Lateral <input type="checkbox"/>	
Colour Options :			
Notes:			

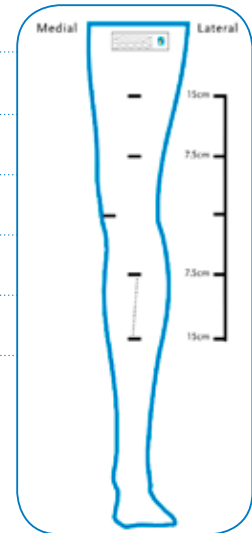

	Anterior	Lateral
File Name: Left		
Right		

Satisfaction Guarantee - Össur brand knee braces are designed and tested to provide the highest level of patient comfort, compliance and effectiveness. If for any reason the patient is not 100% satisfied with their Össur custom fabricated or prefabricated knee brace, the provider may return it within 30 days for a refund.

Product Warranty - Custom Made Knee Braces – Össur warrants the hinge components to be free of defects in material and workmanship for the lifetime of the patient. With regard to components of the brace that wear with use, which include straps, liners, covers, sleeves, return cords and hinge coatings, these components shall be warranted free from defect in material and workmanship for 6 months.

Prefabricated Knee Braces – Össur warrants the frame and hinge components to be free of defects in material and workmanship for two years. With regard to components of the brace that wear with use, including straps, liners, covers, sleeves, return cords and hinge coatings, these components shall be warranted free from defect in material and workmanship for 6 months.
 Return Policy - Upon receipt of the Össur knee brace, the provider shall inspect it to ensure that it meets the requirements set forth in the order. If the brace does not meet the order requirements, the provider must call the Össur Customer Care Department at 08450 065 065, Monday to Friday, 8:30am to 5:00pm. Prior to returning a brace for any reason, including product warranty or patient satisfaction issues, the provider is required to obtain a Return Authorisation (RA) number from Customer Care. Final acceptance of all returned items remains the discretion of the Össur Quality Control Department.

Satisfaction Guarantee and Product Warranty time frames begin on the date the brace is delivered to the patient. Proof of brace delivery, signed and dated by the patient, is required for processing of guarantee and warranty returns. Satisfaction Guarantee does not apply to the Össur Post-Operative braces.

The stabilising effect of a knee brace will depend on the type of brace, patient age, height, weight, muscle tone, severity of the injury/OA, the type of injury and the medical treatment involved. A knee brace is designed to reduce the potential and severity of a knee injury and is not a guarantee against injury. There are no warranties expressed or implied made by Össur on their braces except the warranties against defects in materials or workmanship. Liability is limited to replacement or repair of defective parts or workmanship. Liability for incidental or consequential damage, personal injury or property damage as a result of defect is specifically denied. Össur neither assumes nor authorises any other person to assume for it any other liability for its braces.

